

## Navigating the Touchscreen



### Tap

- Any button to select or deselect
- Trackpad to lock measurement calipers



### Double Tap

- 2D to exit all other modes
- An exam on the Patient List to review



### Swipe

- Gain and Cine bar; side to side
- Depth and Focus bar; flick up or down



### Tap + or -

- for +/- single unit steps
- Gain bar to adjust image brightness
- Cine bar to go through imaging frames



### Slide

- TGC
- Trackpad when measuring



### Tap and Slide

- Tap COLOR; tap and slide trackpad to adjust ROI box size. Tap to set size
- Slide trackpad to move ROI box

### Controls and Buttons

- Active are green
- Inactive are light grey
- Not available are dark grey

## Power Button

You can turn the system on or off using the power button on the scanning engine.



### Input Area

### Basic Touchscreen Buttons

## Begin an Exam

You can start an exam without a transducer connected to the system:

- 1 Tap **PATIENT** to enter patient details, then tap **START EXAM\***.
- 2 Plug in a sterile transducer. The system will automatically activate the transducer in the started exam.

To select a transducer and a preset before you start the exam:

- 1 Enter patient details and tap **NEXT**.
- 3 Select a transducer connected to the system.
- 2 Select **EXAM TYPE** and **Preset**.
- 4 Tap **START EXAM**.


\*If a transducer is connected, the system will automatically go to the default exam type and the default preset for this transducer

### 1 Begin an Exam

Follow Steps 1-2 on page 1.


### 2 Color Mode

Tap **COLOR** to activate. Tap **POWER** to toggle Power Doppler on/off.

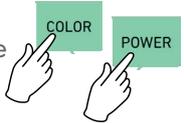


### 3 PW Doppler Mode

Tap **PW** to activate Pulsed Wave Doppler mode. Tap **AUTO** to optimize PW Doppler. Tap **PW** to turn off.

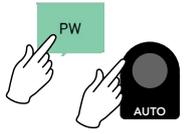

### 4 Needle Guide for Biopsy

Tap **BIOPSY** button to activate/deactivate.

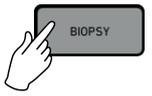

### 5 Capture

Tap **STORE** to capture still frame. Or tap **Freeze**, select **Cine Frame**, then tap **STORE**. Tap **CLIP** to capture a clip.



### 6 Review Exam

1. Tap **REVIEW**. Current exam images will appear.
2. If no exam is open then double-tap on desired exam from the list.



### 7 End an Exam


Tap **END EXAM**.



### Compare Modes\*


#### Dual Live Compare

Tap **Dual Live Compare** to compare live image with previously saved images (active when images were saved during the exam).

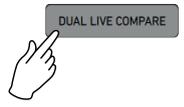

#### Picture-in-Picture Live Compare

Tap **Picture-in-Picture Live Compare** to compare live image to stored images or clips.

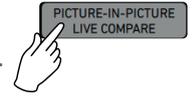

\*Available with license

### Helpful Tips

#### Gain

Tap +/- or slide on the **Gain** bar to adjust.



#### Depth/Focus

Flick up or down on the **Depth/Focus** bar to adjust.



#### Home

Tap **Home** to bring image back to default settings.



#### Measure and Calculate

Tap **Freeze**. Tap **CALC** for calculations. Tap **MEASURE** for Generic measurements. Use tap and slide to position calipers.



#### Label an exam

Tap **LABEL**. Then tap desired entry type:



#### Needle Enhance

Locate and tap **Needle Enhance**. Select **RIGHT** or **LEFT** to indicate needle entry point.



Refer to the User Guide 16-126278 for full instructions.



0543

BK Medical ApS, Mileparken 34, 2730 Herlev, Denmark. T +45 4452 8100 F +45 4452 8199

[REF] 2300  
16-129030-EN-01  
2021-03

