

## Navigating the Touchscreen



### Tap

- Any button to select or deselect
- Trackpad to lock measurement calipers



### Double Tap

- 2D to exit all other modes
- An exam on the Patient List to review



### Swipe

- Gain and Cine bar; side to side
- Depth and Focus bar; flick up or down



### Tap + or - for +/- single unit steps

- Gain bar
- Cine bar



### Slide

- TGC
- Trackpad when measuring



### Tap and Slide

- Tap Color; tap and slide trackpad to adjust ROI box size. Tap to set size
- Slide trackpad to move ROI box

## Power Button

You can turn the system on or off using the power button located on the monitor or on the scanning engine (top-left corner).



## Exam Management

## Transducer

## Current User

## Settings

## Input Area

## Basic Touchscreen Buttons

## Begin an Exam

- 1 Enter patient details and tap **NEXT**
- 2 Select a transducer connected to the system
- 3 Tap **VIEW DETAILS** for the performance test report (US only)
- 4 Select Exam Type and Preset
- 5 Tap **START EXAM**


You can also start the exam immediately after entering patient details. Tap **START EXAM**. Select transducer using the transducer control button. The system will use the default exam type and preset.

### 1 Begin an Exam

Follow Steps 1-4 on page 1.


### 2 Color Mode

Tap COL to activate. Tap P to toggle Power Doppler on/off.

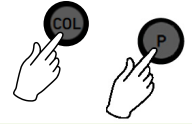


### 3 PW Doppler Mode

Tap PW to activate. Tap TRIPLEX to toggle Triplex off/on.

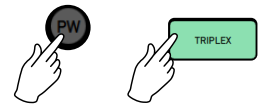

### 4 Measure and Calculate

Tap Freeze.\* Tap CALC for calculations. Tap MEASURE for Generic measurements. Use tap and slide to position calipers.

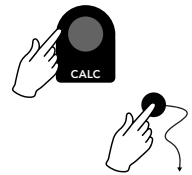

\*CALC can be auto-selected on FREEZE

### 5 Label an Exam

1. Tap LABEL. Then tap desired entry type:




### 6 Capture

Tap STORE to capture still frame. Or tap Freeze, select Cine Frame, then tap STORE. Tap CLIP to capture a clip.



### 7 Needle Guide for Biopsy

Tap BIOPSY button to activate/deactivate.

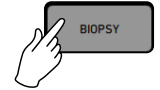

### 8 Review Exam

1. Tap REVIEW. Current exam images will appear.
2. If no exam is open then double-tap on desired exam from the list.




### 9 Create Report

Tap REPORT to create, preview, print and store a report. Images are automatically added to the report.




### 10 Pause an Exam

Tap PAUSE EXAM.

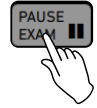

### 10 End an Exam

Tap END EXAM.

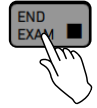

### Helpful Image Adjustments

#### Gain

Tap +/- or slide on the Gain bar to adjust.



#### Depth/Focus

Flick up or down on the Depth/Focus bar to adjust.



#### Home

Tap home to bring image back to default settings.



#### Needle Enhance

Locate and tap Needle Enhance. Select RIGHT or LEFT to indicate needle entry point.



Refer to the bkSpecto User Guide 16-01507 for full instructions.

bkSpecto = [REF] 1300  
16-122545-EN-03  
Issued 2022-02

